PONTE GIULIO®

ISTRUZIONI DI MONTAGGIO - ASSEMBLY INSTRUCTIONS -
MONTAGEANLEITUNGEN - INSTRUCTIONS DE MONTAGE -
INSTRUCCIONES PARA EL MONTAJE

IMPUGNATURE DI SOSTEGNO RIBALTABILI - FOLDING SAFETY
SUPPORT - STÜTZKLAPPGRIFF - BARRES DE SOUTIEN
RELEVABLE - ASIDERO ABATIBLE - ø33 mm

SERIE MAXIMA
G41

 **PRODOTTO CONFORME ALLA - PRODUCT CONFORM TO -
DIESES PRODUKT ERFÜLLT DIE ANFORDERUNGEN DER
EUROPÄISCHEN RICHTLINIE - PRODUIT CONFORME A
LA - PRODUCTO CONFORME A LA
MDD93/42 EEC**

 **ANTIBATTERICO - ANTIBACTERIAL -
ANTIBAKTERIELL - ANTIBACTERIEN - ANTIBACTERIAL**

 **PRODOTTO RICICLATO E RICICLABILE - RECYCLABLE-
PRODUCT - WIEDERVERWERTBARES UND
RECYCLEBARES PRODUKT - PRODUIT RECYCLE ET
RECYCLABLE - PRODUCTO RECICLADO Y RECICLABLE**

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APPLICATION FIELD

THE DEVICE HAS BEEN DESIGNED AND BUILT AT THE SOLE PURPOSE OF FACILITATING THE MOVEMENT OF ELDERLY OR DISABLED PEOPLE. THE DEVICE CAN SUPPORT A MAXIMUM WEIGHT OF 1471 N (150 KG). FOR A CORRECT USE, THE DEVICE MUST BE KEPT FREE FROM ANY OBJECTS AND MUST BE IN A PLACE EASY TO REACH. ROOM TEMPERATURE -5°C + 40

SAFETY WARNINGS

THE DEVICE MUST BE INSTALLED BY QUALIFIED PERSONNEL, WHO MUST CHOOSE THE MOST ADEQUATE FASTENING ELEMENTS FOR THE VARIOUS WALLS. INSTALLERS MUST ENSURE THAT, IN CORRESPONDENCE WITH THE ENTIRE FASTENING FLANGE, THE WALL MUST BE PERFECTLY FLAT, WITH NO JOINTS OR PROTRUDING TILES AND THAT THE WALL STRUCTURE IS SUITABLE FOR FASTENING THE DEVICE AND THERE ARE NO WEAK POINTS. 4 PLUGS MUST BE USED FOR FASTENING FLANGE. THE LACK OR INEFFICIENCY OF ONE OR MORE PLUGS DANGEROUSLY REDUCES THE LOAD RESISTANCE OF THE DEVICE. TIGHT UP THE LOCKING NUTS (FIG.5) TILL THE DEVICE REMAINS IN A VERTICAL POSITION AND DOES NOT FOLD DOWN AND MAY BE LOWERED WITH A SLIGHT EFFORT.

NOTA * MAINTENANCE

CHECK PERIODICALLY (AT LEAST EVERY SIX MONTHS) TIPPING THE MOVEMENT. IT IS POSSIBLE TO ADJUST THE LOUGHDNESS OF THE MOVEMENT BY TIGHTENING UP THE LOCKING NUTS (FIG.5). PONTE GIULIO IS NOT RESPONSIBLE FOR AN INSTALLATION AND/OR TIGHTEN UP OF THE LOCKING NUTS THAT ARE NOT CARRIED OUT AS MENTIONED

CLEANING

USE A DAMP CLOTH DIPPED IN NEUTRAL LIQUID SOAP AND/OR OTHER SANITISING PRODUCTS BASED ON QUATERNARY AMMONIUM SALTS AND SURFACTANTS. DO NOT USE ABRASIVE LIQUID DETERGENTS WITH CHLORINE, STRONG ACIDS OR BASES, CONCENTRATED SOLVENTS, SUCH AS ACETONE AND TRICHLOROETHYLENE AND DERIVATIVES THAT DO NOT IMPROVE ACCESSORY ASEPTICITY, WHICH CAN OPACIFY AND DETERIORATE MATERIALS. DO NOT USE WARM AIR SOURCES FOR DRYING

DISPOSAL

THE USER IS RESPONSIBLE FOR DISPOSAL, WHICH MUST BE CARRIED OUT IN COMPLIANCE WITH THE LAWS OF THE COUNTRY OF INSTALLATION. THE DEVICES ARE NOT MANUFACTURED WITH SUBSTANCES THAT ARE HARMFUL OR HAZARDOUS TO HEALTH AND THE ENVIRONMENT. ALL THEIR COMPONENTS ARE RECYCLABLE AND/OR REUSABLE. IN ORDER TO PROTECT THE ENVIRONMENT, THE DEVICE IS MANUFACTURED WITH PERCENTAGES OF RECYCLED MATERIALS.

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